

Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days ¹

Control	Lot	
Tri-Level	22507-1K	2025-11-27
Low	22507-11	2025-11-27
Normal	22507-12	2025-11-27
High	22507-13	2025-12-01

SWELAB

Swelab Alfa/Alfa Plus		CONTROL	L
		LOT	22507-11
		Mean ²	± Range ³
RBC	10 ¹² /L	2.25	0.12
MCV	fL	67.1	5.0
RDW	%	14.2	4.0
Hct	%	15.1	2.5
Plt	10 ⁹ /L	81	15
MPV	fL	11.5	1.5
WBC	10 ⁹ /L	3.4	0.3
Hgb	g/dL	5.5	0.3
MCH	pg	24.4	1.5
MCHC	g/dL	36.4	3.0
Lymph#	10 ⁹ /L	1.6	0.4
Mid#	10 ⁹ /L	0.1	0.3
Gran#	10 ⁹ /L	1.7	0.8
Lymph	%	47.3	7.0
Mid	%	4.1	5.0
Gran	%	48.6	7.0

CONTROL		N
LOT		22507-12
Mean ²	± Range ³	
4.15	0.18	
81.2	5.0	
13.3	4.0	
33.7	3.0	
245	30	
11.7	1.5	
8.3	0.6	
11.6	0.4	
28.0	1.5	
34.4	3.0	
4.2	0.6	
0.3	0.5	
3.8	1.0	
49.9	5.0	
4.1	5.0	
46.0	5.0	

CONTROL		H
LOT		22507-13
Mean ²		± Range ³
4.91		0.22
92.4		5.0
12.1		4.0
45.4		3.5
554		60
11.9		1.5
20.1		1.8
15.5		0.5
31.6		1.5
34.1		3.0
12.7		1.5
0.6		1.0
6.8		2.5
63.0		5.0
3.1		5.0
33.9		5.0

- 1
- bg Трайност на отворен флакон 14 дни (bg)
 - hr Stabilitet otvorene bočice 14 dni (hr)
 - cs Stabilita otevrené ampulky 14 dny (cs)
 - da Åben rør stabilitet 14 dage (da)
 - et Stabiilsus avatud vialis 14 päevade (et)
 - fr Stabilité en flacon ouvert 14 jour (fr)
 - de Stabilität geöffneter Flaschen 14 tage (de)
 - el Διατήρηση δείγματος μετά την αποσφράγιση 14 ημέρα (el)
 - it Stabilità della fiala aperta 14 giorni (it)
 - lv Atvērt flakonu stabilitāti 14 dienas (lv)
 - lt Stabilitumas atidarius buteliuką 14 dienas (lt)
 - no Åpen rør/glass stabilitet 14 dager (no)
 - pl Trwałość po otwarciu fiolek 14 dni (pl)
 - pt Estabilidade após abertura do frasco 14 dias (pt)
 - ro Termenul de valabilitate al fiolei desfăcute 14 zile (ro)
 - ru Стабильность вскрытого флакона 14 дней (ru)
 - sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
 - sk Stabilita otvorenej injekčnej liekovky 14 dní (sk)
 - sl Stabilita otvorenej injekčneje liekovky 14 dni (sl)
 - es Estabilidad de la cápsula abierta 14 días (es)
 - sv Hållbarhet för öppnad flaska 14 dagar (sv)
 - tr Açık şişe stabilitesi 14 gün (tr)

- 2
- Средно
 - Sredstvo
 - Průměr
 - Middelverdi
 - Keskmine
 - Moyenne
 - Mittelwert
 - Μέση
 - Media
 - Nožimě
 - Vidurkis
 - Middelverdi
 - Średnia
 - Média
 - Medie
 - Среднее
 - Srednja vrednost
 - Priemer
 - Povprečno
 - Media
 - Medelvärde
 - Ortalama

- 3
- Обхват
 - Raspon
 - Rozsah
 - Område
 - Vahemik
 - Intervalle
 - Bereich
 - Αναμενόμενο εύρος α
 - Intervallo
 - Diapazons
 - Intervalas
 - Område
 - Zakres
 - Intervalo
 - Interval
 - диапазон
 - Opseg
 - Rozsah
 - Območje
 - Intervalo
 - Intervall
 - Aralığı

For Swelab Alfa / Alfa Plus with barcode reader

Alfa:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

Alfa Plus:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: Swelab Alfa

Lot: 22507-11

Prod.: Con-Diff Low

ID: 2250711+

Exp.: 27-Nov-2025

Instr.: Swelab Alfa

Lot: 22507-12

Prod.: Con-Diff Norm

ID: 2250712+

Exp.: 27-Nov-2025

Instr.: Swelab Alfa

Lot: 22507-13

Prod.: Con-Diff High

ID: 2250713+

Exp.: 1-Dec-2025

1:



B o u l e C B A V 3



B o u l e C B A V 3



B o u l e C B A V 3

2:



2250711+



2250712+



2250713+

3:



25112700001153030000



25112700002153030000



25120100003153030000

4:



20014018000000000000



20014018000000000000



20014018000000000000

5:



22512671500550315125



41518812501160433730



49122924501550545435

6:



08115115152441536430



24530117152801534430



55460119153161534130

7:



03403016040010301708



08306042060030503810



20118127150061006825

8:



14240473700415048670



13340499500415046050



12140630500315033950

9:



11117395650596114532



11114630095749636147



11115935438413114880